

Rev: A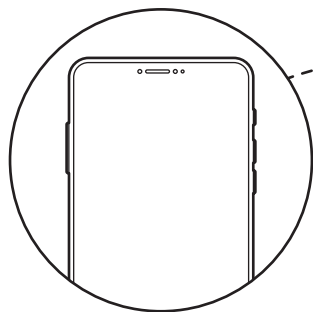

do more™

taskaprosthetics.com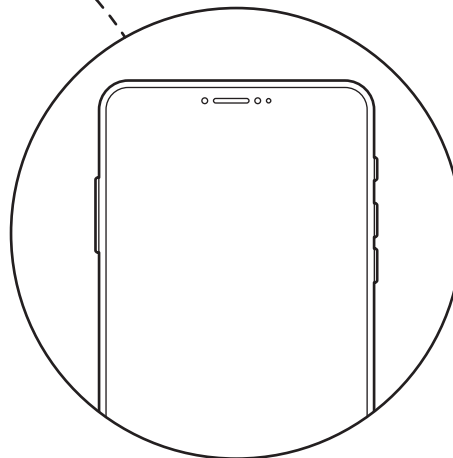

MyTASKA

Remote Connection Clinician Guide

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Please scan this QR code for more helpful TASKA resources.

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EN

Welcome

Remote Connection is a feature within MyTASKA for iOS and Android that enables remote support between clinicians and their patients. It allows clinicians to offer effective remote assistance through the MyTASKA app, reducing the need for patient travel and streamlining the troubleshooting process.

This guide offers step-by-step instructions for setting up and administering remote sessions with Remote Connection, along with troubleshooting tips to help address common issues. Its purpose is to help users maximise the software's capabilities and achieve remote access and management.

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EN

Before you get started

Before initiating a Remote Connection session, first establish clear communication with your hand-connected user (user). Ensure that your user has their TASKA Hand, prosthesis, and device with the latest version of MyTASKA installed. Additionally, your user will need to pair their TASKA Hand to MyTASKA using Bluetooth® and follow the steps below for their model of TASKA Hand:

TASKA HandGen2

Activate Bluetooth by holding down the Hand Button (👉) for 1 second (long press). Using MyTASKA, follow the prompts on screen to pair with the hand.

TASKA CX

Activate Bluetooth (🔗) pairing mode by holding down the Lock Button for an extra long press (4 seconds). Using MyTASKA, follow the prompts on screen to pair with the hand.

Connection steps

The following information applies to the clinician view:

1. Open MyTASKA.

2. From the menu, select 'Remote Connection'.
3. Start the Remote Connection guided walkthrough.
4. A code will be generated. Share this code with your user.
5. To navigate to Remote Connection, instruct your user to follow these steps:
 - Go to settings.
 - Scroll to the last option, select 'Remote Connection'.
 - Enter the code provided by your clinician.
 - Accept on-screen prompt to start the Remote Connection session.
6. The session will now be in progress, indicated by a green bar at the top of the MyTASKA app.

To end a session, press 'End Session' from the prompt on your screen.

Users can also end the Remote Connection session by pressing 'End Session' from the prompt on their screen.

Refer to the troubleshooting section in this guide for further assistance.

For hand specific troubleshooting, the TASKA Hand. refer to the guides supplied with

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Dépannage

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EN

PROBLEM	WHAT TO DO
Trouble connecting hand to MyTASKA.	Refer to the guides supplied with the TASKA Hand.
Unable to see the Remote Connection option in their app.	Note: Remote Connection is not currently available for MyTASKA on Windows. For MyTASKA on iOS and Android, users access Remote Connection (in MyTASKA) after connecting the hand through the settings cog. Clinicians access Remote Connection through the MyTASKA menu. Ensure both you and the user have the latest version of MyTASKA.
Remote Connection not available from the MyTASKA menu.	Note: Remote Connection is not currently available for MyTASKA on Windows. For MyTASKA on iOS and Android users access Remote Connection (in MyTASKA) after connecting the hand through the settings cog. Clinicians access Remote Connection from the MyTASKA menu. If Remote Connection is not showing, request clinician access by contacting your distributor or emailing support@taskaprosthetics.com
Unable to establish Remote Connection.	Ensure the TASKA Hand is connected to MyTASKA. Ensure both you and your user have the latest version of MyTASKA. Check the internet connection for both you and your user.
Settings made remotely not applying to the hand.	Ensure your user's MyTASKA is not hidden behind another application. Check the internet connection for both you and your user. Restart the Remote Connection for both you and your user.

Contact and support

Contact your TASKA representative for further troubleshooting assistance.

For hand-specific troubleshooting, refer to the guides supplied with the TASKA Hand.

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