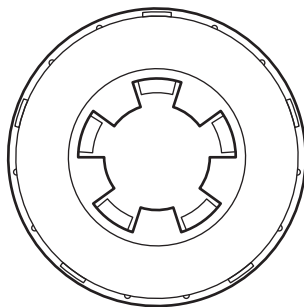
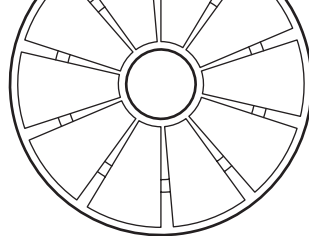
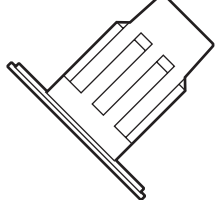
Rev: F

do more™
taskaprosthetics.com

TASKA QDW

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do more™

EN

The Lamination Collar uses the industry-standard quick disconnect interface, which can connect to various terminal devices. When paired with a compatible TASKA hand and Seal Ring, it creates a waterproof wrist connection between the hand and prosthesis. See our website for compatibility.

The interface between the Lamination Collar and our Coaxial Plug is unique to TASKA, meaning the collar cannot be fitted with other plugs.

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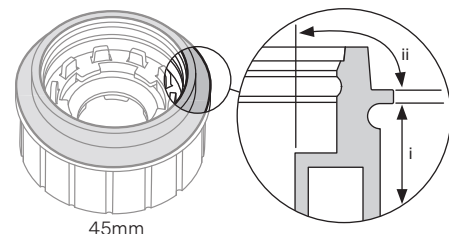
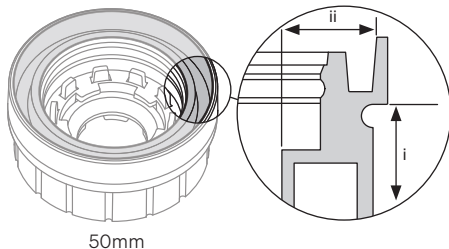
50mm

EN Installing Lamination Collar.

Assemble Lamination Collar and masking components. Ensure there are no gaps between the components. Use the Lamination Bolt to fasten to an alignment jig if required, tightening to seal in place.

Laminate using the preferred method of fabrication, or, glue in place.

Glue method:

Glue the collar in place using a structural resin. Ensure good coverage around the perimeter, as any gaps may affect the waterproof seal in this area.

Lamination method:

Use the preferred method to laminate into place.

Ensure a good bond around the lamination area (i).

Masking tape may be used on the outermost edges of the Lamination Collar to achieve a smooth finish.

During the grinding phase, begin by grinding down until the bolt head is exposed. Remove the bolt.

Carefully grind down to plastic masking cap radially, taking care to not grind into sealing areas or locking features (ii). Doing so could permanently damage the collar.

Remove Masking Plug and other lamination accessories and discard.

⚠ Avoid damaging or getting glue/resin on the sealing surface or the locking features (ii) of the Lamination Collar. Doing so may compromise the strength and waterproof seal of the wrist connection and will require replacement of the collar.

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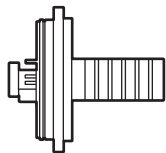
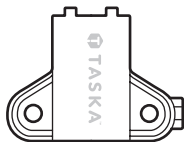
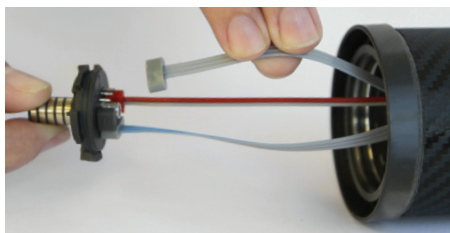
do more™

EN

Installing Coaxial Plug. Please refer to the images on pages 43 and 44 to assist you.

Components:

1. Coaxial Plug Assembly (includes Blanking Plugs)

Tools required:

2. Coaxial Plug Installation Tool

Installation

3. Check that the Lock Ring is properly seated on the Coaxial Plug.

Remove the Retaining Nut using the drive feature on the Coaxial Installation Tool handle. Discard unrequired Blanking Plugs from the back of the Coaxial Plug.

4. Feed electrode and power cables through the opening of the installed Lamination Collar.

Use a small bead of Silicone Grease, as recommended by the sensor system manufacturer, on each of the Coaxial Plug pins. This is to protect the plug connection from sweat or excessive moisture.

Plug the cables into the appropriate pins on the Coaxial Plug (see diagram on page 41) and remove any extra grease.

5. Refit Retaining Nut, gently tightening with the Installation Tool.
6. Insert the Coaxial Plug and Lock Ring into the Lamination Collar.
7. Use the Coaxial Installation Tool to secure Lock Ring. Rotate Clockwise until it clicks into place. Attach a TASKA hand to check the functionality of the connections.

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 **TASKA™**

EN

- TASKA does not accept responsibility for the waterproofing between the Lamination Collar and the socket.
- If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent authority of the country you live in.
- Lamination Collars have a lifetime of 5 years.
- This product is an electrical device and should not be disposed of with household waste. Please contact your local authority for return and collection information.

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