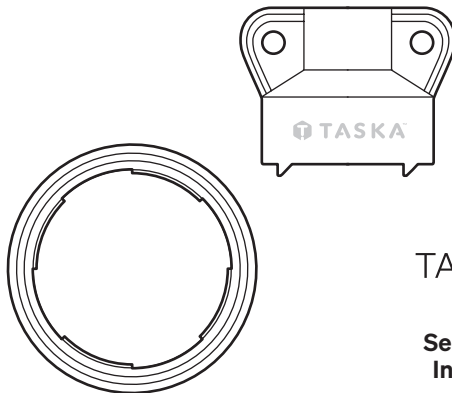
Rev: E

TASKA QDW

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do more™

EN

The Quick Disconnect TASKA hand may be attached to a range of wrist connectors or electric rotators, however not all of these connections can be made waterproof. A waterproof wrist connection requires a Seal Ring to be fitted to the hand, with the matching wrist connector installed into the socket.

Seal Rings are only compatible with appropriate Quick Disconnect TASKA hands and selected wrist connectors or electric rotators. Refer to our website for compatibility information.

Sealing surfaces must be maintained and checked regularly to ensure a waterproof connection is maintained. Care instructions are included in the User Guide.

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3

2

EN Installing the sealing ring. Please refer to the images on pages 21 and 22 to assist you.

1. Check the surfaces on the Seal Ring and where it attaches to the hand for dirt or damage prior to installing. If required, clean the surfaces with a soft, damp cloth.
2. Connect the Seal Ring by slotting it onto the hand and rotating clockwise.
3. Tighten using the Seal Ring Installation Tool until you feel and hear it click into place.

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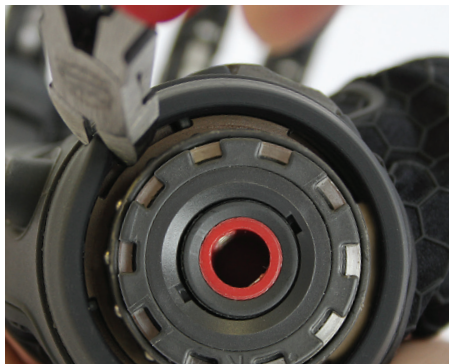
1

2

EN Removal. Please refer to the images on page 31 to assist you.

1. Use a pair of side-cutters (not provided) to cut through the rigid plastic inner of the Seal Ring.
2. Once the rigid plastic inner has been cut, the soft remainder of the Seal Ring may be torn apart, and the ring removed.
3. Follow the instructions for installing the new Seal Ring.

⚠ Take care to avoid scratching the metal surfaces of the hand. Deep scratches on the metal surface may compromise the waterproof seal.

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Varningar

EN

- Ensure your patient follows all instructions and adheres to all warnings in the TASKA QDW Seal Ring & Lamination Collar User Guide.
- If either the Seal Ring or sealing surface on the wrist connector/electric rotator is damaged (torn, cracked, split etc) it must be replaced.
- If your patient experiences a slowing or stalling wrist rotation when using a powered wrist rotator, some lubricant may need to be added. A small bead of the following Silicone based lubricants can be used:
 - Super-Lube Multi-Purpose Grease with Syncolon (PTFE)
 - Ottobock Silicone Grease 400, medium (item # 633F11)

This should be applied to the interior surface of the Seal Ring, or the running surface of the rotator at signs of slowing.

- If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent authority of the country you live in.
- Seal Rings have a 1 year lifetime and must be replaced at annual servicing of the TASKA hand.

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