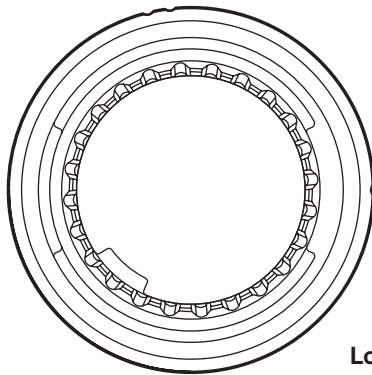
Rev: C

**TASKA CX™**

## Low Profile Clinician Guide

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Svenska SV

EN

The TASKA Low Profile Wrist is a lightweight, waterproof alternative to the Quick Disconnect Wrist. With a lower build height and a fully waterproof connection, it is designed for patients with long residual limbs, such as wrist disarticulation and transradial limbs, or those who have no need to change terminal devices. The Low Profile Wrist is unique to the TASKA Hand and is available in all TASKA Hand sizes.

Please ensure you read and understand the information contained in the TASKA CX Hand Clinician Guide. Refer to the TASKA Learning Academy for further compatibility information.

#### Key features:

- 300° rotation
- 21 positions
- will not disconnect unexpectedly (Prosthetist fitting and removal only)
- allows for long residual limbs.

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SK

SV

This image shows a single sheet of white paper with horizontal blue or grey ruling lines. The lines are evenly spaced and run across the width of the page. There are approximately 20 lines visible. The paper has a slight shadow on the right side, suggesting it's part of a bound notebook.

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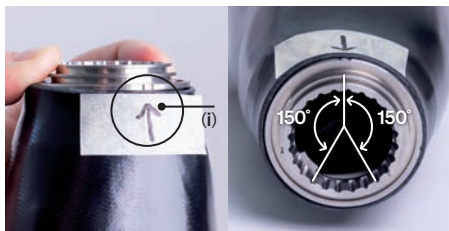
EN

EN

1. Hold the hand in the position defined as the centre of available rotation. Create an **orientation reference mark** on the socket which aligns to the palm centre of the hand (i).
2. Using your **orientation reference mark** (i), align the single notch (right hands) or double notch (left hands, shown in the example) that appear on the Lamination Collar.
3. Temporarily secure the collar in place, and test available range of motion by attaching the hand (refer to the **Install** section). Adjust the centre of rotation if required.
4. Remove the hand from the collar and re-install the lamination accessories before laminating the final socket.

- ⚠ Do not laminate the wrist with the hand attached, and always use the lamination accessories to protect the mechanical features of the collar from contamination or damage.
- ⚠ Verify the dorsal alignment of the hand and socket is correct prior to laminating. Incorrect lamination will likely result in re-fabrication of the forearm. See the lamination procedure for further details.

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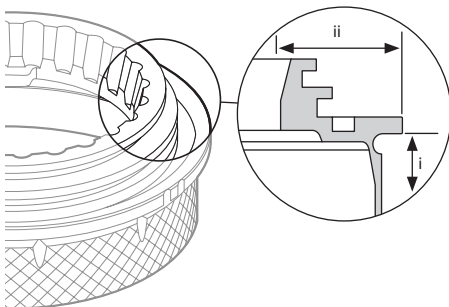
do more™

### EN Lamination procedure.

Assemble the Lamination Collar to the lamination and masking accessories. Ensure the flat seal (Masking Gasket), is seated in the groove. If installed incorrectly, there may be a large gap.

Check the orientation is correct from the previous section.

Laminate using the preferred method of fabrication, or, glue in place.

#### **Lamination method:**

Use the preferred method to laminate into place.

Ensure a good bond around the lamination area (i). Masking tape may be used on the outermost edges of the Lamination Collar to achieve a smooth finish.

During the grinding phase, begin by grinding down until the bolt head is exposed. Remove the bolt.

Carefully radially grind down to the plastic Masking Cap. Take care to not grind into the sealing areas or locking features (ii), as doing so could permanently damage the collar.

Remove the Masking Cap and other lamination accessories and discard.

#### **Glue method:**

Glue the collar in place using a structural resin. Ensure good coverage around the perimeter, as any gaps may affect the waterproof seal.

Do not grind into the metal of the Lamination Collar! This may damage the sealing surfaces or mechanical features of the collar.

- ⚠** Installing the O-ring is critical to the waterproof connection. This must be performed prior to installation of the hand.

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do more™

**EN Installation.** Please refer to the diagram and images on pages 61 and 63 to assist you.

1. Follow the lubrication and cleaning instructions outlined in the **Care** section of this guide to install the O-ring.
2. Make the electrical connections according to the diagram on page 61. Failure to do so may damage the electronics in the hand.
3. Position the palm side of the hand 180 degrees out from the orientation reference mark (refer to the **Orientation** section).
4. Press the hand firmly in, and rotate in either direction to engage the lock (at least 2 clicks).
5. Test the lock by turning the hand back in the opposite direction. It should stop against the lock and remain connected.

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do more™

**TASKA™**

**EN Removal.** Please refer to the images on page 75-76 to assist you.

1. Rotate the hand until it stops against the lock.
2. Peel back the hem of the fabric boot back from the wrist on the palm side, under the thumb.
3. Using the TASKA Screwdriver, or other flat blade screwdriver, pull up on the mushroom shaped plunger.
4. While holding the plunger up, continue to rotate the hand in same direction as step 1. Rotate 2 clicks.
5. Pull hand away from the socket.

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6

7

**EN Lubrication and cleaning.** Please refer to the images on pages 85-87 to assist you.

Cleaning and lubricating the running surfaces of the Low Profile Wrist must be performed annually, alongside hand servicing. It is also recommended every time the hand is removed after extended use. This is to ensure the best possible user experience with the product and to help maintain the waterproof seal and functionality of the wrist rotation.

1. Carefully remove the O-ring from the Lamination Collar and remove any dirt or old lubricant using a cloth and denatured alcohol or dish detergent and water. Other chemicals could contaminate the plastic or socket material and cause damage.
2. Remove excess dirt and dust from the mechanical surfaces of the Lamination Collar as shown.
3. Apply Low Profile Wrist Lubricant to the O-ring groove, retention grooves, and detent feature on the Lamination Collar.
4. Check the O-ring for damage or degradation to the smooth surface.
5. Lubricate the O-ring using the provided Low Profile Wrist Lubricant. Coat evenly to create a light film around the entire O-ring.
6. Re-install the O-ring by pushing it down into the annular groove.

Additional Low Profile Wrist Lubricant can be used to stick the O-ring down in the groove.

7. Using a clean rag only, clean the interface plate on the hand. Remove residual grease and dirt.

**▲** Inspect and re-lubricate the sealing O-ring on the socket side once per year, at the time of hand service. If there are any signs of wear, tear, degradation, or damage, it should be replaced. Contact your TASKA representative for information on ordering additional O-rings.

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Varningar

## EN

- All TASKA Hands have a waterproof rating of IP67 up to the wrist. Please discuss further waterproofing with your clinician. TASKA Prosthetics is not responsible for the waterproofness of the prosthetic socket that your TASKA Hand is attached to.
- If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent authority of the country you live in.
- The lifetime of the TASKA CX is 5 years when serviced annually.
- Lamination Collars have a lifetime of 5 years.
- This product is an electrical device and should not be disposed of with household waste. Please contact your local authority for return and collection information.

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