Rev: C

do more™
taskaprosthetics.com

TASKA™
by eqwal

EMERGO EUROPE
Westervoortsewijk 60,
6827 AT Arnhem
The Netherlands

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Svenska SV

EN

The TASKA PowerCore system (also referred to as 'the system') is designed for powering a myoelectric terminal device, and the components incorporated into a prosthesis, to assist you in your Activities of Daily Living (ADLs). Please read, understand, and follow these instructions on how

to use your system safely and appropriately. Failure to do so may cause damage to you, others, and/or the system, and could invalidate your warranty.

The TASKA PowerCore system must be installed by a trained and certified clinician.

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LED display overview

Charging

For LED display during charging please refer to the charging section of this guide.

Power on/off (long press >0.5 second)

LED DISPLAY	LED COLOUR & BEHAVIOUR	POWER STATE
	White flashes once	On
	White flashes twice	Off

Display battery level (short press <0.5 second)

LED DISPLAY	LED COLOUR & BEHAVIOUR	BATTERY LEVEL %
	Amber flashes 3 times	<10
	Green illuminates	20
		40
	Green LEDs illuminate one by one to show battery level	60
		80
		100

Alarms

LED DISPLAY	LED COLOUR & BEHAVIOUR	ACTION REQUIRED
	Amber flashes continuously	The battery has no remaining power - stop using prosthesis and charge immediately.
	Red illuminates continuously	Fault, perform a reset (4 second extra long press on power button), wait 10 seconds, then power on (>0.5 second long press on power button). If fault is still present contact your clinician.

If there are no LEDs displayed and the system is unresponsive, in the first instance connect your charger. If the system is still unresponsive, contact your clinician.

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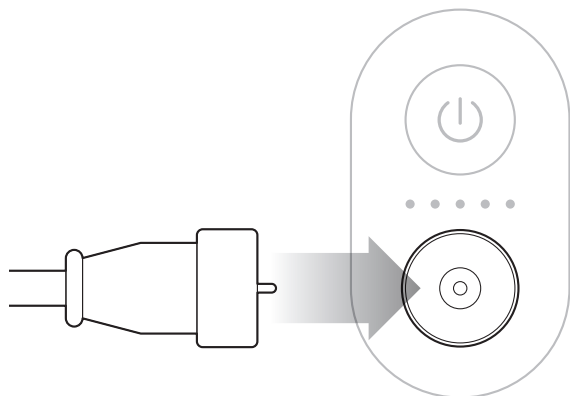
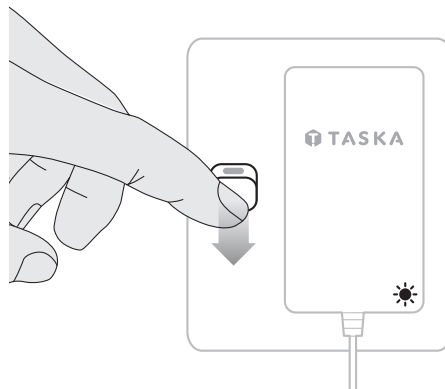
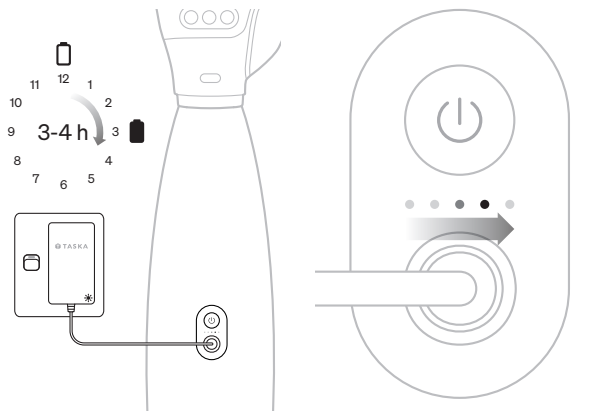
TASKA™

EN

Charging. Please refer to the diagrams on pages 39-40 to assist you.

LED display on power switch

LED DISPLAY	LED COLOUR & BEHAVIOUR	POWER STATE
	Green animated sequence (figure 5)	Charging in progress
	Green all illuminated	Charging complete

LED display on charger

LED COLOUR & BEHAVIOUR	POWER STATE
Red illuminates continuously	Charging in progress
Green illuminates continuously	Charging complete

- Detach the prosthesis from the residual limb while charging.
- Check charger label for compatible operating limits.
- Ensure that the mains power is turned off when connecting the charger to the switch.

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do more™

EN

The lifetime of the system is 5 years, when cared for as recommended in this guide.

The system must be inspected regularly by your clinician for damage and sweat ingress.

Battery care

To achieve the best results, it is recommended to:

- charge every night;
- charge before storing for long periods;
- do not let the system become completely flat.

Talk to your clinician if you feel that the battery performance has degraded.

TASKA recommends replacing the battery(s) after 12 months of regular use.

Chemicals

Do not expose your system to corrosive substances such as solvents, acids, alkalis, strong detergents, industrial chemicals, and any substances that are harmful to human skin. For example, acetone, bleach or turpentine.

Waterproofing

This is dependent on the components and installation methods used by your clinician.

Speak with your clinician for limitations regarding water exposure.

Cleaning

Clean the power switch when visibly dirty, after exposure to chemicals or as desired, by wiping it with a soft damp cloth, soap or light disinfectant. Avoid harsh chemicals as they may damage the device.

Salt water may discolour the charging socket on the power switch. If exposed during an activity, wipe down with a soft cloth dampened with fresh water.

The magnetic connector on the charging socket may attract iron filings. To remove these, use a sticky substance such as tape or wax.

Environmental conditions

Operating & storage temperature range: -10°C to 45°C (14°F to 113°F)
Charging temperature: 0°C to 45°C (32°F to 113°F)

Atmospheric pressure: 75kPa to 111kPa (10.9PSI to 16.1PSI)

Operating humidity:
5 - 100% relative humidity,
non-condensing

Storage humidity:
45 - 85% relative humidity,
non-condensing

Input voltage: 12-24 V DC
Input current: 1 A max

Specifications

Battery capacity: 2000mAh
Nominal voltage: 7.4 V
Maximum output current: 5 A
System weight: 105 g
Mains charger (TASKA-MC-01)
Model: CH0161-0841000F
Input: mains/grid
Input voltage: 100-240V AC
Input frequency: 50-60 Hz
Input current: 0.5 A max
Car charger (TASKA-CC-01)
Input: automotive auxiliary power outlet

Compatibility

Speak with your clinician regarding compatible terminal devices.

Usable Extinguishing Agent

AVD, K36 and Lith-EX.

Disposal

This product is an electrical device with a lithium-ion (Li-Ion) battery and should not be disposed of with household waste. Please contact your local authority for return and collection information.

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Varningar

EN

Please read, understand, and follow these warnings on how to use your system safely and appropriately. Failure to do so may cause damage to you, others, and/or the system, and could invalidate your warranty.

- Turn your system off before you attach or detach a terminal device to your socket, failure to do so may cause a fault.
- Do not try to repair or modify the system yourself. Repairs or modifications by anyone not authorised by TASKA Prosthetics will invalidate the warranty.
- The system will automatically disable power preventing use of the prosthesis in the following scenario's: when the battery has no remaining power, while charging, in the event of a fault.
- Do not wear your prosthesis while charging.
- For charger input power compatibility refer to the label on the provided charger. Do not connect either charger to a power supply which does not meet these specifications.
- The system has the potential for interference with cardiac implants. Both the charging socket and charger have a magnetic connection. If you have a cardiac implant, you must keep these parts a safe distance away from your implant as recommended by the safety information supplied with your cardiac implant.
- Do not charge the system outside of the specified environmental operating conditions.
- Only use a TASKA Charger and TASKA PowerCore Switch to charge the system.
- Do not charge the system in the presence of flammable liquids or gas.
- Do not change or short circuit the battery wires.
- Do not expose the battery to naked flames.
- Do not bend, pierce, or damage the case of the battery.
- Shipping of Li-Ion batteries is regulated, when transporting individual batteries, make sure that the battery terminals are protected and well insulated from materials that could contact them and cause a short circuit. Confirm the regulations for transportation, packing and shipment with your shipping carrier before transporting Li-Ion batteries.
- If the battery is leaking, swollen or unusually hot. Immediately stop using or charging prosthesis and contact your clinician.
- If the system shows visible signs of damage, send for repair.
- If a serious incident occurs in relation to this device, it should be reported to TASKA Prosthetics and the competent authority of the country you live in.

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