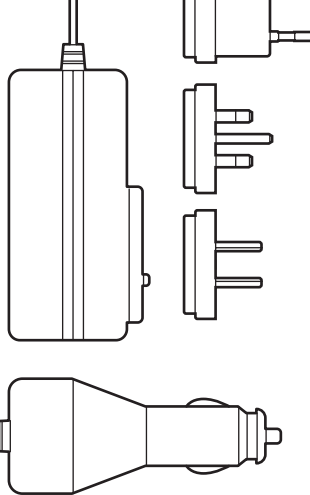
Rev: C

do more™

taskaprosthetics.com

T TASKA™

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 **TASKA™**

EN

The TASKA Power System is designed for powering a myoelectric terminal device in order to assist in performing activities of daily living.

Please observe the following to understand how to use the system safely and appropriately.

Ensure your user is familiar with the warnings and instructions in the TASKA Gen1 Power System User Guide.

The TASKA Power System must be installed by a trained and certified clinician.

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EN

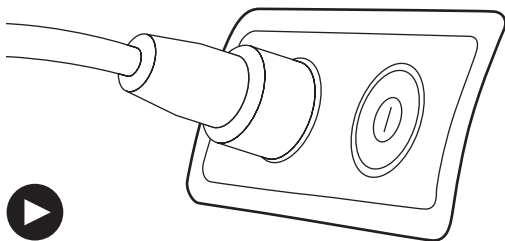
The power button turns the TASKA Hand (or other terminal device) on or off.

TASKA Battery charging process

1. Doff prosthesis.
2. Connect charger to power source. See specifications for charger operating limits.
3. Connect the magnetic tip of the charger's lead to the magnetic

charging socket on the Power Switch (see figure 1).

4. The light on the mains or car charger will change from green to red to show that charging is in progress.
5. When charging is complete, the light on the charger will return to green.

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Varningar

Battery warnings

- Do not charge the battery outside of the environmental operating conditions specified.
- Do not expose the battery to naked flames.
- Do not bend, pierce, or damage the case of the battery.
- If the battery is leaking, swollen, or unusually hot; immediately stop using or charging the prosthesis and replace the batteries.
- Do not expose to temperatures below -10°C (14°F) or above 60°C (140°F).

Power System warnings

- TASKA Prosthetics is not responsible for the waterproofness of the prosthetic socket or Power System. Please ensure the user is aware of the waterproof limitations of their socket. This will likely be the first opening on the prosthesis.
- Contact your TASKA representative if something is wrong with the Power System.
- If the Power System is broken or shows visible signs of damage, please contact your TASKA representative for repair options.
- Do not try to repair or modify the Power System yourself. Repairs or modifications by anyone not authorised by TASKA Prosthetics will invalidate the warranty.
- Do not change or short circuit the battery wires.
- Only use a TASKA Charger and Power Switch to charge TASKA Batteries.

- Do not charge TASKA Batteries in the presence of flammable liquids or gas.
- Dispose of batteries and other components according to your local regulations.

Specifications

Operating voltage: 7.4V nominal
Maximum current: 5.5A
Battery weight: 88.6g
Mains charger input voltage: 100-240V AC
Mains charger input frequency: 50/60Hz
Car charger voltage: 10-30V DC

TASKA Power System environmental conditions

Charging temperature: 0°C to 45°C (32°F to 113°F)
Operating temperature range: -10°C to 45°C (14°F to 113°F)
Storage temperature range: -10°C to 35°C (14°F to 95°F)
Operating and storage humidity range: 5 to 90% relative humidity, non-condensing

Compliance and compatibility

Visit taskaprosthetics.com for up-to-date compliance and compatibility information.

If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent authority of the country you live in.

For further information on the TASKA Power System, ensure you're signed up

and registered for the **TASKA Learning Academy**.

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