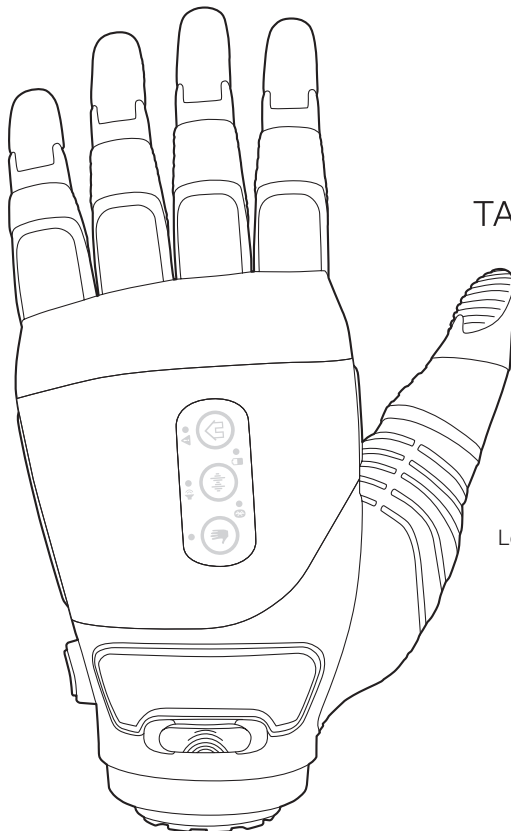
do more™
taskaprosthetics.com

TASKA
by eqwal

TASKA HandGen2

Clinician Guide

Vejledning til kliniker
Anleitung für
Orthopädietechniker
Manual del clínico
Lääkärin opas
Manuel clinicien
Leiðbeiningar meðferðaraðila
Guida per lo specialista
Handleiding voor clinici
Veiledning for leger
Guia do médico
Руководство для врачей
Príručka pre lekárov
Läkarhandbok

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Svenska SV

EN

The TASKA hand is a myoelectric prosthetic hand intended to assist with activities of daily living. Please read, understand, and ensure these instructions are followed on how to use a TASKA

hand safely and appropriately. Failure to do so may cause damage to users and/or the hand, and could invalidate the warranty.

TASKA hand's must be installed by a trained and certified clinician.

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Varningar

do more™

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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EN

When considering a TASKA hand the following aspects should be taken into account:

- appropriate sizing of the device
- ability of the device to meet user requirements
- ability of the user to accept the weight of the device when incorporated into the overall prosthesis
- ability for the user to control the device.

In order to ensure optimal outcomes for the user, these aspects should be reassessed after

fitting and setup is completed.

Ensure the user is familiar with the following warnings and instructions. This information can also be found in the **TASKA HandGen2 User Guide**.

Visit taskaprosthetics.com for a full list of requirements and compatibility for sensors, batteries, chargers and power systems.

For further information on the TASKA hand, ensure you're signed up and registered for the **TASKA Learning Academy**.

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preludes and instructions. This information can also be found in the **HandGen2 User Guide**.

A full list of requirements and information for compatibility with sensors, batteries, chargers, and power systems.

devices and feeding systems can be viewed on the website taskaprosthetics.com.

To receive additional information about the TASKA prosthesis, you must register on the **TASKA Learning Academy** platform.

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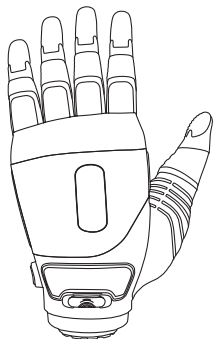
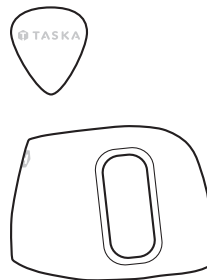
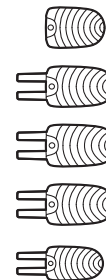
Using your hand

- Do not rely solely on your TASKA hand to support your weight, or as the primary means to prevent a fall.
- The maximum weight you may carry with your TASKA hand or any other prosthetic hand connected to a TASKA Wrist is 20 kg.
- Do not use your TASKA hand for holding anything that could cause harm if dropped, such as a heavy or fragile object or container holding a hazardous substance.
- Your TASKA hand is intended to assist you in your activities of daily living. It is not designed to be used for activities that involve a lot of vibration, impact, or force to the hand. Examples of inappropriate use include, but are not limited to:
 - using high-impact tools such as hammers, impact wrenches, or hammer drills
 - using heavy-duty machinery such as chainsaws and reciprocating saws
 - deliberately hitting the hand against hard surfaces
 - weightlifting
 - high-intensity, adventure, or contact sports.
- Do not expose the TASKA hand to hazards that would injure a natural hand, for example:
 - heavy impact
 - severe vibration
 - explosions
 - corrosive chemicals
 - dangerous machinery
- naked flames or anything hot enough to burn human skin.
- Use your TASKA hand carefully to avoid pinching skin, injuring people or damaging objects.
- Use of a prosthetic arm can cause body strain, discomfort or irritation to the residual limb. Talk to your clinician if your TASKA hand is causing pain or discomfort.
- Before operating any vehicle with your TASKA hand, make sure you are legally allowed to operate the vehicle with a prosthesis. You must obey local regulations when using vehicles, aircraft, watercraft, or other forms of motorised transport while using your TASKA hand. Please talk to your clinician for further information.
- If use of your TASKA hand is allowed while operating a vehicle:
 - use an open, static grip so that it does not accidentally close on a control
 - do not use your TASKA hand to operate safety-related controls such as a brake
 - make sure the battery has enough charge to last the length of the journey.
- Do not use your TASKA hand in a way that could cause a safety hazard.
- Do not use your TASKA hand to operate firearms.
- Do not expose your TASKA hand to live electrical wiring.
- Do not use your TASKA hand in the presence of flammable liquids or gas.

- If your TASKA hand is fitted with a Seal Ring, clean this and the sealing surface on your socket's wrist connection with a soft damp cloth to remove any dirt before connecting.
- You must return your TASKA hand for its annual service in order to maintain the warranty. You may be invoiced for costs that are incurred as a result of the hand not receiving its annual service.
- If the hand stops responding, try turning the hand off and on again.
- Contact your clinician if something is wrong with your hand.
- If you notice lubricants or other fluids leaking out of the TASKA hand, contact your clinician immediately for repair and cleaning.
- Do not try to repair or modify your TASKA hand yourself. Dismantling of your hand beyond removal of defined user replaceable components by anyone not authorised by TASKA Prosthetics will invalidate the warranty of your TASKA hand.
- If your TASKA hand is broken or shows visible signs of damage, please contact your prosthetist for repair options. Continuing to use your hand may result in compromising the waterproof seal. Note that damage to the fabric thumb boot or faceplates does not affect the waterproofness of your hand.

Chemicals

Do not expose your TASKA hand to corrosive substances such as solvents, acids, alkalis, strong detergents, industrial chemicals, and any substances that are harmful to human skin.

Washing your TASKA hand

Clean your TASKA hand regularly only using water, soft soap, or light disinfectant to wash the hand. Harsh chemicals may damage the hand.

Always wash your TASKA hand after using it in situations that could make it dirty (including salt water). Clean the hand before using it to eat, prepare food, or treat injuries.

Risk of unpredictable movement from electromagnetic interference

Operating any myoelectric hand or device, including the TASKA hand, near strong electromagnetic fields **may cause unpredictable movements**. This interference can disrupt the myoelectric sensors that control your prosthesis, leading to unintentional opening, closing, or other erratic motions.

Although exposure to these sources may not cause harm to the TASKA hand in the short term, extended exposure or extremely high strength magnetic fields can permanently damage the TASKA hand.

Maintain a safe distance from known sources of significant electromagnetic interference or switch the hand off when within these environments. Interference may arise from, but is not limited to:

- X-ray machines
- therapeutic medical machines (e.g., radiotherapy)
- overhead power lines
- induction cooktops and microwave ovens
- radio and telecommunication antennas.

If you experience unpredictable movements, **move away from the**

suspected source of interference, or turn the hand off.

The **radiation exposure from airport X-ray machines is minimal** and not harmful to prosthetic devices or users, but it is recommended that the hand be powered off.

Please consult your local authority around security requirements when travelling with a prosthetic device.

Environmental conditions

Operating & storage temperature range: -20 °C to 60 °C (4 °F to 140 °F)

Operating & storage humidity:
5 - 90 % relative humidity,
non-condensing

Operating your TASKA hand outside the operating range may result in unpredictable behaviour.

When not using your TASKA hand we recommend storing it in a dry environment in the protective case provided.

Waterproof

All TASKA hands have a waterproof rating of IP67 up to the wrist. Please discuss further waterproofing with your clinician. TASKA Prosthetics is not responsible for the waterproofness of the prosthetic socket that your TASKA hand is attached to.

Glove use

It is recommended a glove is used on the TASKA hand for non-hazardous

tasks that would normally require the use of a glove.

Power consumption

It is recommended the TASKA hand is used with a TASKA Power System. Peak performance and battery life may be affected when using a 3rd party power system.

Software

The MyTASKA app allows you to connect to your TASKA hand via Bluetooth to adjust and customise the settings and grips. To download the software, please use the following link: taskaprosthetics.com/support

Compliance and compatibility

Visit taskaprosthetics.com for up-to-date compliance and compatibility information.

If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent authority of the country you live in.

Lifetime

The lifetime of the TASKA hand is 5 years when serviced annually.

Disposal

This product is an electrical device and should not be disposed of with household waste. Please contact your local authority for return and collection information.

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