do more™  
taskaprosthetics.com

**TASKA™**  
by eqwal

**TASKA CX™**

## User Guide

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Svenska SV

EN

The TASKA hand is a myoelectric prosthetic hand intended to assist you in your activities of daily living. Please read, understand, and follow these instructions on how to use your TASKA hand safely and appropriately. Failure to do

so may cause damage to you, others, and/or the hand, and could invalidate your warranty.

**Your TASKA hand must be installed by a trained and certified clinician.**

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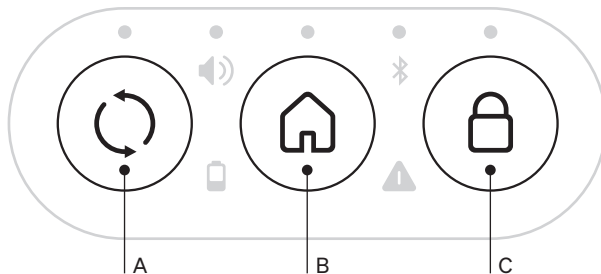
SV

## Button panel overview

| A - CYCLE                                                        | B - HOME                               | C - LOCK                        |
|------------------------------------------------------------------|----------------------------------------|---------------------------------|
| <b>SHORT PRESS (&lt;1 SECOND)</b>                                |                                        |                                 |
| Cycle through up to 5 grips (default are Flexitool, Tripod, Key) | Home Grip 1 (default is General Grasp) | Lock/unlock device              |
| <b>LONG PRESS (&gt;1 SECOND)</b>                                 |                                        |                                 |
| Audio on/off                                                     | Home Grip 2 (default is Pincer)        | Bluetooth® on/off               |
| <b>EXTRA LONG PRESS (&gt;4 SECONDS)</b>                          |                                        |                                 |
| -                                                                | -                                      | Activate Bluetooth pairing mode |

### Power

Your hand will automatically turn on when the power is supplied. Please refer to the relevant user manual for power system operation. As your hand turns on, lights on the button panel will flash briefly and

the fingers will open for calibration. Ensure the fingers are free to open fully and calibrate correctly.

### Warning lights

If the warning light (▲) comes on, there is a problem with the hand. Turn the power off and on

again (power cycle). If the light still appears, contact your clinician. The Low Battery light (■) will flash if there is less than 10 % battery life remaining.

### Bluetooth

Bluetooth (✱) can be turned on and off by holding down the Lock Button for 1 second (long press).

### MyTASKA app

All grips and selected sensor settings are user configurable using the MyTASKA app. The app can be downloaded to your Android or iOS device from the appropriate app store. Follow the on screen instructions and refer to the TASKA website for further information.

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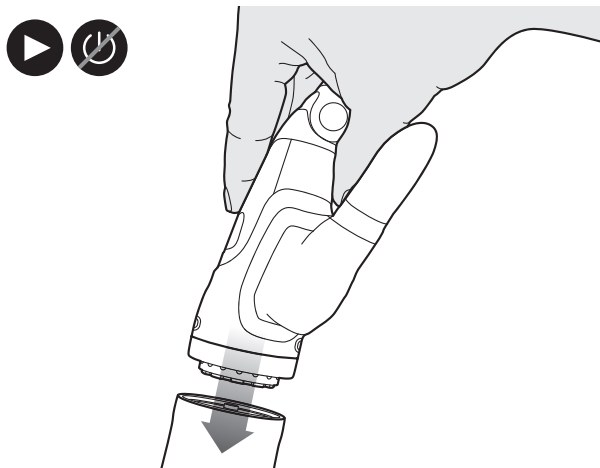
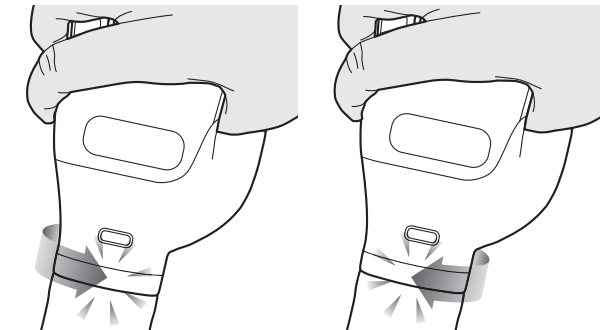
3

EN

**Attaching your TASKA hand.** Please refer to the diagrams on page 49 to assist you.

If your TASKA hand has a Quick Disconnect (QD) Wrist, follow these steps to **connect**.

**Make sure your socket is turned off before you attach your TASKA hand.**

1. Insert your TASKA hand into the wrist connection on your socket.

2. Rotate your hand towards you 45° until it clicks into place.

3. Test the connection by rotating your TASKA hand in the other direction. The wrist should rotate normally, making a clicking sound, and feel secure.

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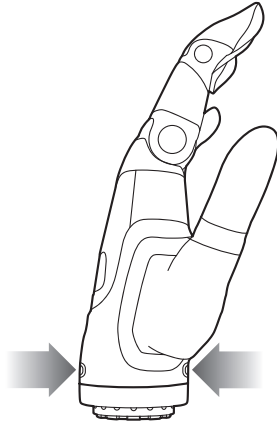
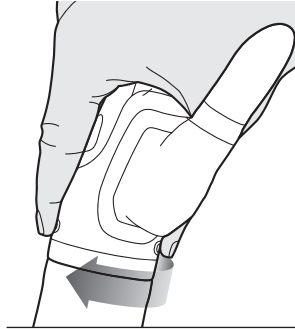
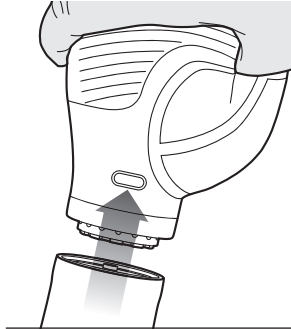
3

EN

**Detaching your TASKA hand.** Please refer to the diagrams on page 59 to assist you.

If your TASKA hand has a Quick Disconnect (QD) Wrist, follow these steps to **disconnect** using the release buttons.

**Before removal, make sure your TASKA hand is turned off using the power button on your socket.**

1. To remove your TASKA hand, press and hold down both release buttons.
2. Rotate the hand away from you 45° until you feel resistance. This turn will be smooth and without audible clicks.
3. Remove your TASKA hand.

If your TASKA hand has a Low Profile (LP) Wrist, you cannot attach or remove it from your socket. Only a trained clinician may install and remove it. Do not attempt to attach or remove the hand yourself.

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This image shows a single page of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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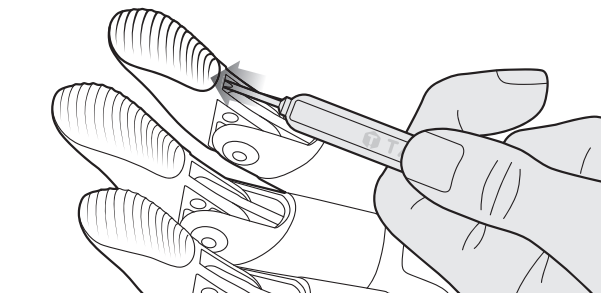
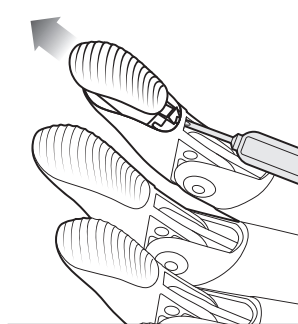
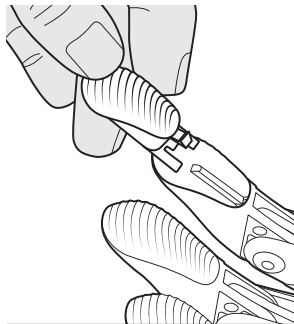
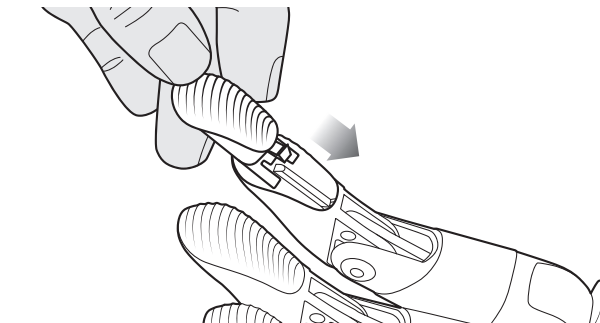
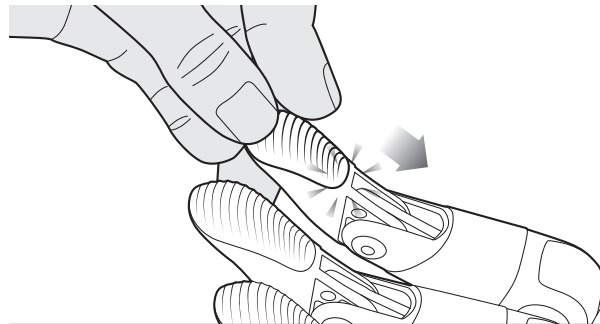
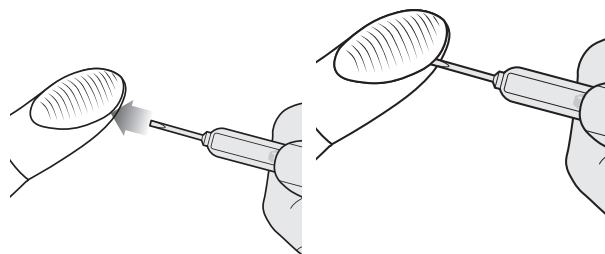
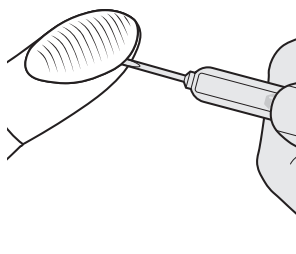
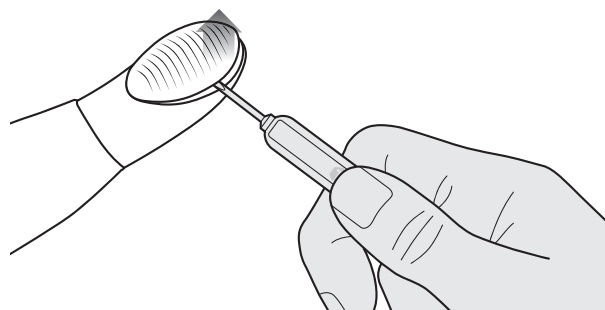
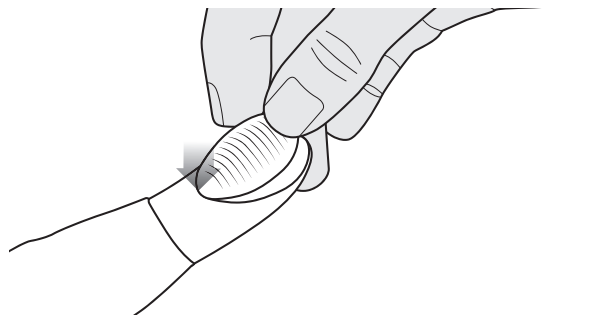
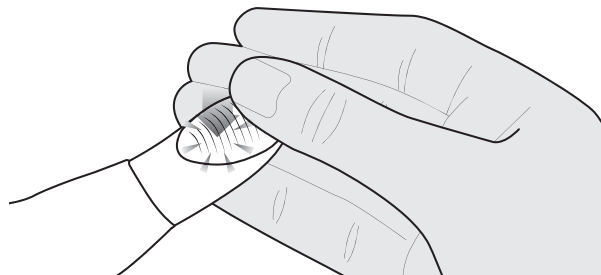
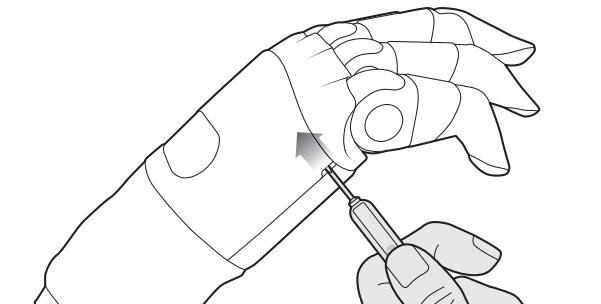
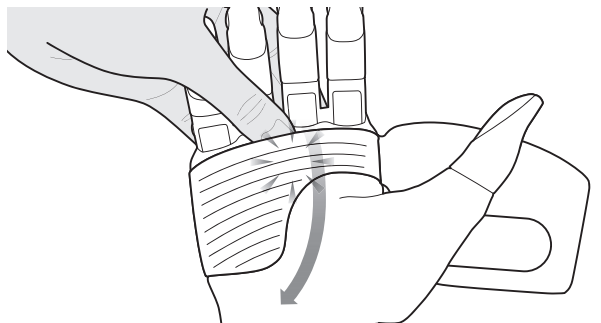
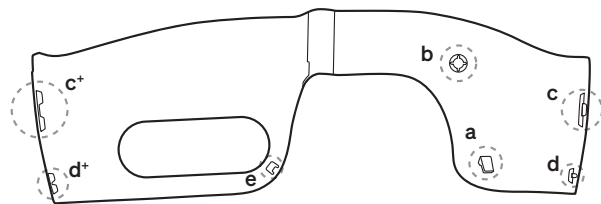
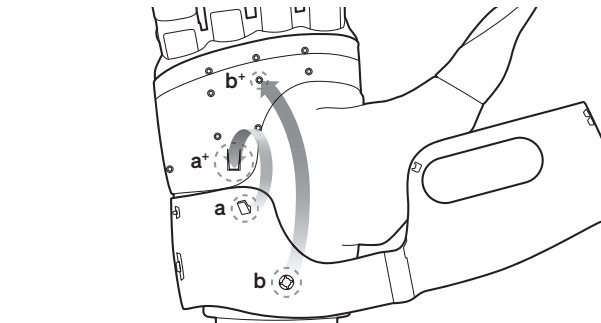
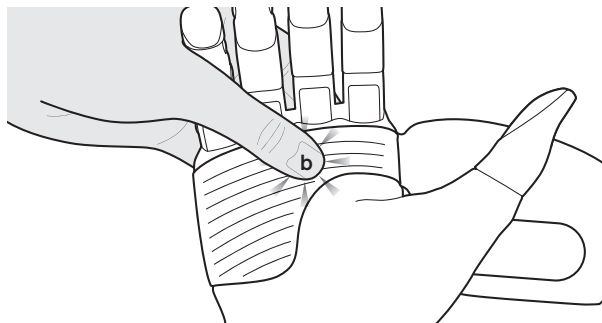
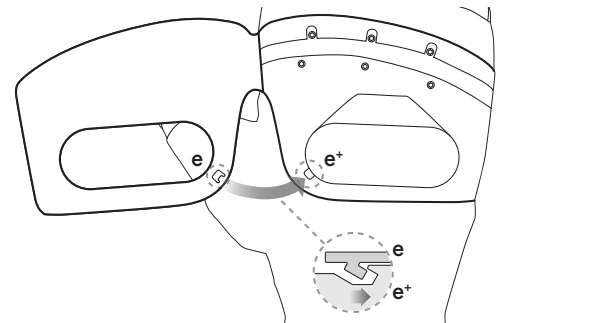
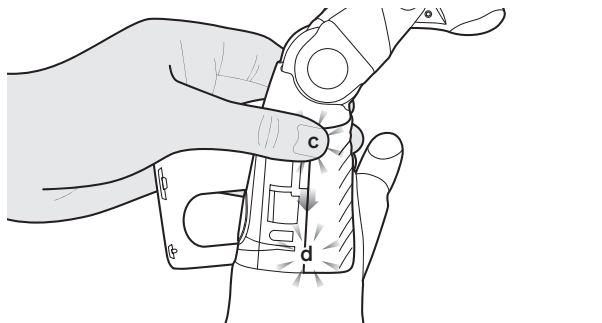
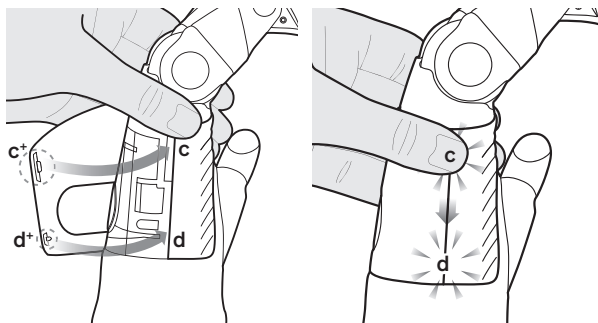
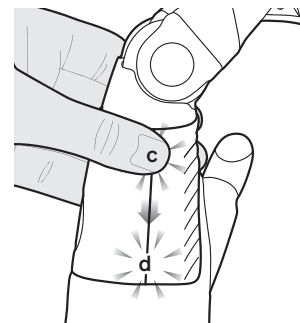
## EN

**Using your hand**

- Do not rely solely on your TASKA hand to support your weight, or as the primary means to prevent a fall.
- The maximum weight you may carry with your TASKA hand or any other prosthetic hand connected to a TASKA Wrist is 20 kg.
- Do not use your TASKA hand for holding anything that could cause harm if dropped, such as a heavy or fragile object or container holding a hazardous substance.
- Your TASKA hand is intended to assist you in your activities of daily living. It is not designed to be used for activities that involve a lot of vibration, impact, or force to the hand. Examples of inappropriate use include, but are not limited to:
  - using high-impact tools such as hammers, impact wrenches, or hammer drills
  - using heavy-duty machinery such as chainsaws and reciprocating saws
  - deliberately hitting the hand against hard surfaces
  - weightlifting
  - high-intensity, adventure, or contact sports.
- Do not expose the TASKA hand to hazards that would injure a natural hand, for example:
  - heavy impact
  - severe vibration
  - explosions
  - corrosive chemicals
  - dangerous machinery
- naked flames or anything hot enough to burn human skin.
- Use your TASKA hand carefully to avoid pinching skin, injuring people or damaging objects.
- Use of a prosthetic arm can cause body strain, discomfort or irritation to the residual limb. Talk to your clinician if your TASKA hand is causing pain or discomfort.
- Before operating any vehicle with your TASKA hand, make sure you are legally allowed to operate the vehicle with a prosthesis. You must obey local regulations when using vehicles, aircraft, watercraft, or other forms of motorised transport while using your TASKA hand. Please talk to your clinician for further information.
- If use of your TASKA hand is allowed while operating a vehicle:
  - use an open, static grip so that it does not accidentally close on a control
  - do not use your TASKA hand to operate safety-related controls such as a brake
  - make sure the battery has enough charge to last the length of the journey.
- Do not use your TASKA hand in a way that could cause a safety hazard.
- Do not use your TASKA hand to operate firearms.
- Do not expose your TASKA hand to live electrical wiring.
- Do not use your TASKA hand in the presence of flammable liquids or gas.

- If your TASKA hand is fitted with a Seal Ring, clean this and the sealing surface on your socket's wrist connection with a soft damp cloth to remove any dirt before connecting.
- You must return your TASKA hand for its annual service in order to maintain the warranty. You may be invoiced for costs that are incurred as a result of the hand not receiving its annual service.
- If the hand stops responding, try turning the hand off and on again.
- Contact your clinician if something is wrong with your hand.
- If you notice lubricants or other fluids leaking out of the TASKA hand, contact your clinician immediately for repair and cleaning.
- Do not try to repair or modify your TASKA hand yourself. Dismantling of your hand beyond removal of defined user replaceable components by anyone not authorised by TASKA Prosthetics will invalidate the warranty of your TASKA hand.
- If your TASKA hand is broken or shows visible signs of damage, please contact your prosthetist for repair options. Continuing to use your hand may result in compromising the waterproof seal. Note that damage to the fabric thumb boot or faceplates does not affect the waterproofness of your hand.

**Chemicals**

Do not expose your TASKA hand to corrosive substances such as solvents, acids, alkalis, strong detergents, industrial chemicals, and any substances that are harmful to human skin.

**Washing your TASKA hand**

Clean your TASKA hand regularly only using water, soft soap, or light disinfectant to wash the hand. Harsh chemicals may damage the hand.

Always wash your TASKA hand after using it in situations that could make it dirty (including salt water). Clean the hand before using it to eat, prepare food, or treat injuries.

**Risk of unpredictable movement from electromagnetic interference**

Operating any myoelectric hand or device, including the TASKA hand, near strong electromagnetic fields **may cause unpredictable movements**. This interference can disrupt the myoelectric sensors that control your prosthesis, leading to unintentional opening, closing, or other erratic motions.

Although exposure to these sources may not cause harm to the TASKA hand in the short term, extended exposure or extremely high strength magnetic fields can permanently damage the TASKA hand.

Maintain a safe distance from known sources of significant electromagnetic interference or switch the hand off when within these environments. Interference may arise from, but is not limited to:

- X-ray machines
- therapeutic medical machines (e.g., radiotherapy)
- overhead power lines
- induction cooktops and microwave ovens
- radio and telecommunication antennas.

If you experience unpredictable movements, **move away from the**

## EN

**suspected source of interference,** or turn the hand off.

The **radiation exposure from airport X-ray machines is minimal** and not harmful to prosthetic devices or users, but it is recommended that the hand be powered off.

Please consult your local authority around security requirements when travelling with a prosthetic device.

#### Environmental conditions

Operating & storage temperature range: -10 °C to 45 °C (14 °F to 113 °F)

Operating & storage humidity: 5 - 90 % relative humidity, non-condensing

Operating your TASKA hand outside the operating range may result in unpredictable behaviour.

When not using your TASKA hand we recommend storing it in a dry environment in the protective case provided.

#### Waterproof

All TASKA hands have a waterproof rating of IP67 up to the wrist. Please discuss further waterproofing with your clinician. TASKA Prosthetics is not responsible for the waterproofness of the prosthetic socket that your TASKA hand is attached to.

#### Glove use

It is recommended a glove is used on the TASKA hand for non-hazardous tasks that would normally require the

use of a glove. Use of a non-TASKA glove may affect performance of your hand.

#### Power consumption

It is recommended the TASKA hand is used with a TASKA Power System. Peak performance and battery life may be affected when using a 3rd party power system.

#### Software

The MyTASKA app allows you to connect to your TASKA hand via Bluetooth to adjust and customise the settings and grips. To download the software, please use the following link: [taskaprosthetics.com/support](https://taskaprosthetics.com/support)

#### Compliance and compatibility

Visit [taskaprosthetics.com](https://taskaprosthetics.com) for up-to-date compliance and compatibility information.

If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent authority of the country you live in.

#### Lifetime

The lifetime of the TASKA hand is 5 years when serviced annually.

#### Disposal

This product is an electrical device and should not be disposed of with household waste. Please contact your local authority for return and collection information.

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