Rev: H

TASKA QDW

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Handledsskötsel

EN

When using a TASKA hand, care must be taken to maintain the waterproof seal. Your Lamination Collar Kit includes a Seal Ring, fitted to your hand, and a sealing surface on the Lamination Collar itself (see diagram on page 11). Regularly check these parts for damage such as tearing or deep scratches which could compromise the seal.

Regularly clean the Seal Ring and sealing surfaces on your Lamination Collar by wiping with a soft, damp cloth.

If using an MC Seal Ring Kit, note that this is only compatible with the MC Forearm Endcap. Please refer to the relevant user guide for maintenance instructions for that component's sealing surface.

If using a wrist rotator and you notice slowing in the wrist rotation speed over time, contact your clinician for advice on lubricating the seal ring.

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Varningar

EN

- TASKA Prosthetics is not responsible for the waterproofness of the prosthetic socket that your TASKA hand will be attached to. Please discuss your socket's waterproofness with your clinician.
- If you notice damage to the Seal Ring or sealing surfaces, the waterproof seal may be compromised. Do not submerge your hand in water beyond the wrist and contact your clinician.
- If there are signs of moisture on the electronics in your hand or Lamination Collar, dry thoroughly before reconnecting the hand. Contact your clinician if moisture appears regularly.
- The maximum weight you may carry with your TASKA hand or any other prosthetic hand connected to a TASKA Wrist is 25 kg.
- Failure to follow these instructions may invalidate your warranty (please see the warranty document for full details).
- The lifetime of the Lamination Collars is 5 years. Seal Rings have a 1 year lifetime and must be replaced at annual servicing of the TASKA hand.
- Visit taskaprosthetics.com for up-to-date compliance and compatibility information. If a serious incident occurs in relation to this device it should be reported to TASKA Prosthetics and the competent authority of the country you live in.
- This product is an electrical device and should not be disposed of with household waste. Please contact your local authority for return and collection information.

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TASKA Prosthetics Ltd